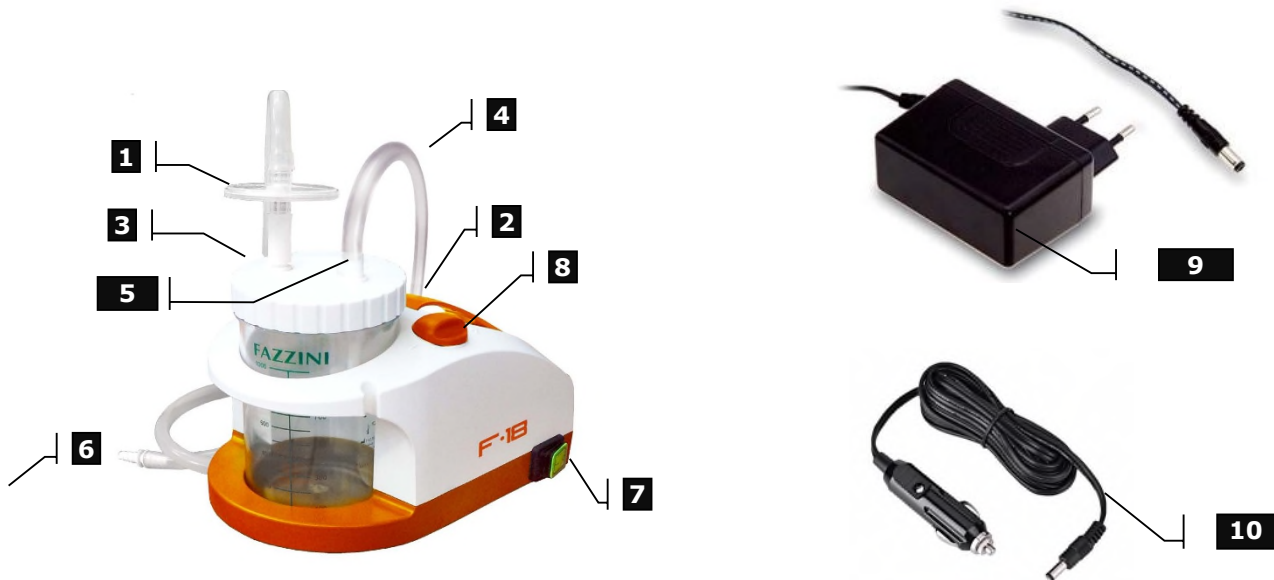
F-18 Battery | **CE** 0123

ASPIRATORE	Istruzioni per l'uso
SUCTION PUMP	Instructions for use
ASPIRATEUR	Mode d'emploi
ASPIRADOR	Manual de instrucciones
ASPIRADOR	Instruções de uso

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INSTRUCTIONS FOR USE

Duty time : 30 min ON / 30 min OFF

VACUUM REGULATOR - Vacuum regulator, identified with orange knob, allows to operate on air circuit. If you rotate the knob clockwise you will increase negative pressure up to maximum suction possible. If you rotate the knob anticlockwise pressure will decrease. During pressure regulation refer to vacuum gauge showing pressure value stated in "bar".

OVER-FLOW DEVICE - Jars covers are supplied with an over-flow device that is made by a float and a support cage. It works, stopping flow transit, when suction liquids reach maximum jar level.

CLEANING AND DISINFECTION OF REUSABLE PIPES AND COMPONENTS At the end of every cycle all the reusable parts (pipes, jars and possible mobile components) must be cleaned. Remove organic material eliminating every residual, disinfect.

INSPECTIONS OF REUSABLE COMPONENTS – All the reusable parts must be checked carefully to localize possible damages, deformations or blocked passages which can prejudice the correct working of the aspirator.

Any deformed element, any crack or deposit of foreign material on any suction unit component has to be replaced. After visual inspection proceed assembling all elements (tubes, antibacterial filter and jars) to check circuit holding placing vacuum regulator at maximum vacuum position. If closing terminal part of aspiration the vacuum gauge indicate maximum negative pressure (declared for that unit) the device is ready for a new cycle.

EXTRAORDINARY MAINTENANCE - In case of accidental aspiration of organic material inside the suction unit stop immediately all operation on the patient. Send back the unit to maintenance service for the replacement of damaged parts.

RESTRICTION OF USE - The unit is intended to be used to aspirate organic fluids during home, hospital or field (when equipped of a rechargeable battery) treatments. It is mandatory to check electric features of the unit and IP protection mentioned on data plate before start using the unit.

ATTENTION

NOT TO ASPIRATE GASES LIKE OXYGEN OR ITS MIXTURES.

THE IMPROPER USE OF THIS DEVICE CAN BE DANGEROUS FOR PATIENT.

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OPERATING MANUAL

WARNINGS
BEFORE USING THE APPLIANCE READ THE HANDBOOK

WARNINGS
THE APPLIANCE MUST BE USED BY QUALIFIED PERSONNEL (NURSES, PHYSICIAN...)

WARNINGS
DO NOT OPEN THIS DEVICE FOR ANY REASON - PLEASE CONTACT THE FAZZINI S.R.L. TECHNICAL DIVISION FOR ANY PROBLEM SHOULD ARISE

CAUTIONS
FOLLOW THESE GUIDELINES IN USING THE INSTRUMENT:
DO NOT CHANGE THE DEVICE, ITS FAILURE TO COMPLY WITH DOOR TO LOSS OF SAFETY AND FAILURES
NOT HANDLE THE DEVICE WITH WET OR DAMP HANDS,
DO NOT HANDLE THE PLUG WITH WET HANDS
DO NOT USE THE DEVICE IN SHOWER OR BATHTUB.
DO NOT IMMERSE THE APPLIANCE IN, NO WATER GETS.
DO NOT USE THE APPLIANCE IN THE PRESENCE OF INFLAMMABLE AND/OR EXPLOSIVE LIQUIDS, GASES OR MIXTURES
DO NOT LEAVE THE APPLIANCE EXPOSED TO ATMOSPHERIC AGENTS OR HEAT SOURCES
DO NOT LET CHILDREN OR INCAPABLE PERSONS USE THE APPLIANCE WITHOUT THE NECESSARY SUPERVISION.
BEFORE STARTING TO CLEAN THE DEVICE, MAKE SURE IT IS NOT CONNECTED TO THE ELECTRIC POWER SUPPLY NETWORK
DO NOT USE THE APPLIANCE FOR AIMS OTHER THAN THOSE FORESEEN BY THE MANUFACTURER. USING THE APPLIANCE UNDER DIFFERENT CONDITIONS THAN THOSE SPECIFIED IN THIS HANDBOOK MAY PREJUDICE ITS SAFETY AND ITS TECHNICAL PARAMETERS
NOT TO USE THE DEVICE NEAR OR ON OTHER DEVICES. IN THIS CASE IS NECESSARY TO LOOK AT THE DEVICE TO CONTROL ITS NORMAL WORKING IN THE CONFIGURATION IN WHICH IT IS USED

BATTERY

WARNINGS
KEEP THE APPLIANCE UNDER CHARGE WHEN NOT IN USE. THE APPLIANCE CAN BE RECHARGED WITH THE APPROPRIATE EXTERNAL CABLE OR BY PLACING THE APPLIANCE ON THE RELEVANT RECHARGE STIRRUP. BEFORE USING IT FOR THE FIRST TIME, IT IS NECESSARY TO KEEP BATTERIES UNDER CHARGE FOR 24 HOURS NON-STOP.

A BATTERY CONSTANTLY UNDER CHARGE LASTS ABOUT 2 OR 3 YEARS. AFTER THIS PERIOD OF TIME IT IS RECOMMENDED TO REPLACE IT.

IF THE APPLIANCE CANNOT BE PUT UNDER CONTINUOUS RECHARGE, RECHARGE IT FOR 24 HOURS EVERY 15 DAYS.

WHEN FINISHED USING IT, THE DEVICE MUST ALWAYS BE RECHARGED. ATTENTION, THE BATTERY MAY BE DAMAGED IF NOT RECHARGED AFTER USE THE BATTERY THAT IS COMPLETELY FLAT AND IS NOT IMMEDIATELY RECHARGED AFTER USE MUST BE REPLACED BY THE AUTHORIZED TECHNICAL ASSISTANCE.

WHEN RECHARGING, ALWAYS MAKE SURE THE WARNING LIGHT ON THE FRONT PANEL IS ON.

1. WHEN ON THE DEVICE'S DISPLAY ONLY THE RED LED IS ON, WHILE THE GREEN LED AND THE TWO YELLOW LED ARE OFF, IT IS NECESSARY TO DISCONTINUE OPERATIONS AND RECHARGE THE DEVICE.
2. TO CHARGE THE DEVICE, KEEP TO THE FOLLOWING INSTRUCTIONS: TO SWITCH POSITION "0", INSERT THE 230V PLUG INTO A WALL SOCKET, THE BLINKING LED SHALL TURN ON AND KEEP CHARGING FOR AT LEAST 12 HOURS.

THE DEVICE CAN OPERATE ON AN EXTERNAL 12 V BATTERY USING THE APPROPRIATE CABLE WITH CAR PLUG. FAZZINI S.R.L. TAKES ALL THE NECESSARY MEASURES TO GUARANTEE THE SAFETY OF ITS PRODUCTS. HOWEVER IT IS IMPORTANT TO COMPLY WITH BASIC SAFETY RULES TO AVOID ELECTRIC DISCHARGES OR FIRES. WHEN USING IT FOR THE FIRST TIME AND THROUGHOUT THE USE OF THE PRODUCT, CHECK THAT THE POWER SUPPLY CABLE DOES NOT SHOW ANY DAMAGE AS TEARS OR PEELINGS, IN WHICH CASE DO NOT PLUG IN THE APPLIANCE TO THE POWER SOCKET.

ON FIRST USE, CHECK THAT THE APPLIANCE SHOWS NO BREAKS, CRACKS OR OPENINGS CAUSED BY TRANSPORTATION, IN WHICH CASE DO NOT SWITCH ON THE APPLIANCE

OPERATING INSTRUCTIONS

PORTABLE SUCTION MACHINE SUITABLE FOR SURGERY. WITH THE EXCEPTION OF THORACIC SURGERY.

SUCTION PUMP FOR BODY LIQUID. FOR OUTPATIENT AND HOSPITAL. NOT FOR HOME USE

OPERATIONAL TEMPERATURE: BETWEEN 0° AND 35° / RECOMMENDED RECHARGE TEMPERATURE: BETWEEN 15° AND 30°

OPERATIONAL HUMIDITY: BETWEEN 20 AND 90% RU

BEYOND 2500 METERS OF HEIGHT, THE ASPIRATION FLOW MAY DECREASE.

THE FOLLOWING STEPS ARE POSSIBLE ONLY IF THE APPLIANCE IS COMPLETELY RECHARGED

1. TIGHTLY SCREW THE LID ON THE JAR TO GUARANTEE A PERFECT SEAL
2. CONNECT THE SHORT TUBE TO FILTER 1 ON THE "aspiration" 2 VENT
3. CONNECT THE OTHER END OF THE TUBE TO THE "vacuum" 3 VENT WITH THE FLOAT ON THE JAR'S LID
4. CONNECT THE LONG TUBE 4 TO THE OTHER "patient" 5 VENT ON THE JAR'S LID (VENT WITHOUT FLOAT)
5. CONNECT THE OTHER END OF THE LONG TUBE 4 TO THE PROBE CONNECTOR 6 AND EVENTUALLY THE SUPPLIED PROBE PRESS THE "I" SWITCH 7

ASPIRATION INTENSITY CAN BE VARIED WITH THE RELEVANT CONTROL ON THE FRONT PANEL OF THE APPLIANCE 8 AND CAN BE CHECKED ON THE APPLIANCE FRONT PANEL VACUOMETER.

IT IS RECOMMENDED TO CARRY OUT A FIRST ASPIRATION CYCLE WITH AT LEAST 500 ml OF WATER ONLY.

THE ASPIRATED FLUID WILL FLOW IN THE RELEVANT JAR. SHOULD THE AMOUNT OF FLUID EXCEED THE JAR'S CAPACITY, THE OVERFLOW VALVE WILL STOP ASPIRATION.


IMMEDIATELY SWITCH OFF THE APPLIANCE BY SETTING THE ON/OFF 7 SWITCH ON THE " OFF" POSITION.

INDICATORS (LED) ONLY FOR MODEL F-18BATTERY

THE SUCTION PUMP F-18 BATTERY IS EQUIPPED WITH LIGHT INDICATORS (LED) SIGNALING THE BATTERY USE, BECAUSE USING THE APPLIANCE ON A LOW BATTERY MAY DAMAGE THE APPLIANCE ITSELF, WHILE CONTINUOUSLY RECHARGING IT DOES NOT CAUSE ANY DAMAGE.

IT IS RECOMMENDED TO ALWAYS CHECK THE BATTERY'S RECHARGE LEVEL

LIGHT INDICATOR	DESCRIPTION OF THE BATTERY LEVEL
LED ON GREEN	MAXIMUM CHARGE (2)
LED ON YELLOW	MEDIUM CHARGE (3)
LED ON YELLOW	LOW CHARGE (4)
LED ON RED	NEEDS RECHARGING (5)
LED NEXT TO THE BATTERY'S CHARGE LEVEL INDICATORS RED	WHEN THIS LED IS ON, IT SHOWS THAT RECHARGE IS UNDERWAY (1)

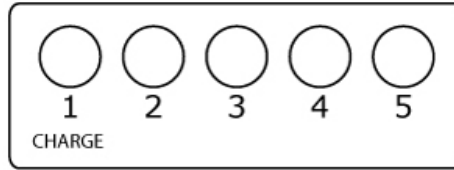THE MANUFACTURER, ASSEMBLER, INSTALLER OR IMPORTER ARE HELD LIABLE FOR THE SAFETY, PERFORMANCE AND RELIABILITY OF THE APPLIANCE ONLY IF THE ELECTRIC SYSTEM TO WHICH THE APPLIANCE IS CONNECTED COMPLIES WITH L.D.46/90

ACCESSORIES INCLUDED

CODE N°	DESCRIPTION
05.0001	JAR 1000 ML
05.00075	SET OF TUBES - AUTOCLAVABLE
05.00552	SET OF TUBES - DISPOSABLE
05.0012	SUCTION PUMP PROBE CH 16 (DISPOSABLE)
05.0016	ANTIBACTERIAL/HYDROPHOBIC FILTER (DISPOSABLE)
SP.38.06	12V ADAPTOR

SHOULD ACCESSORIES BE USED OTHER THAN THOSE SUPPLIED BY FAZZINI S.R.L., YOU MUST MAKE SURE THAT THEY HAVE THE SAME CHARACTERISTICS AS THE ORIGINAL ONES.

NOTE: THE SUPPLIED JARS ARE NOT MEANT TO BE USED AS MEASURERS, AND PROVIDE ONLY AN INDICATION OF THE CONTAINED QUANTITY.

OPERATIONAL STEPS AT THE END OF THE PROCEDURE

NOTE: ALWAYS USE INDIVIDUAL PROTECTION DEVICES (EX. LATTICE GLOVES) EVERY TIME YOU MAY GET IN CONTACT WITH POTENTIALLY CONTAMINATED PARTS.

1. DISCONNECT DISPOSABLE PARTS AND DISPOSE OF THEM.
2. CHECK THE GENERAL INTEGRITY OF THE ASPIRATOR
3. CARRY OUT THE FOLLOWING CLEANING OPERATIONS
4. PUT THE APPLIANCE UNDER RECHARGE

TUBES AND CONNECTORS MUST BE REPLACED FOR ANY PURPOSE.

THE FILTER ON THE APPLIANCE IS DISPOSABLE; THEREFORE IT IS NECESSARY TO REPLACE IT AFTER EVERY USE. DO NOT USE THE ASPIRATOR WITHOUT A FILTER

JAR AND TUBE CLEANING

THE FLUID COLLECTION JAR CAN BE AUTOCLAVED AT A TEMPERATURE NOT EXCEEDING 121 °C FOR NO MORE THAN MAX 3 MINUTES 1 ATM, STEAM OVER THE VESSEL MAY BE DAMAGED. IF EXCEEDED, THE JAR MAY GET DAMAGED OTHERWISE, THE JAR CAN BE DISINFECTED WITH DENATURED ALCOHOL OR A HYPO CHLORITE BASED SOLUTION THAT CAN BE EASILY FOUND IN PHARMACIES MAXIMUM REUSE LIMIT: 30 CYCLES

HOW TO CLEAN THE APPLIANCE

DISCONNECT THE APPLIANCE FROM THE ELECTRIC SUPPLY NETWORK AND THEN CLEAN IT WITH A SOFT CLOTH WETTED WITH NON ABRASIVE DETERGENTS

PERIODIC MAINTENANCE CHECK

THE APPARATUS 'DESIGNED TO REDUCE THE MAXIMUM MAINTENANCE OPERATIONS, BUT YOU HAVE TO DO AN INSPECTION BEFORE USE. CONTROL ALL COMPONENTS (PIPES, TANKS AND ANTIBACTERIAL FILTER) CHECK THE SEAL OF THE CIRCUIT SWITCHING THE FAN WITH FLOW REGULATOR IN POSITION OF VACUUM MAX. IF YOU CLOSE THE TERMINAL PART ASPIRATION THE GAUGE INDICATES THE MAXIMUM DECLARED VOID THE UNIT IS READY FOR A NEW USE

BEFORE MAKING ANY TRANSACTION VERIFICATION IN THE EVENT OF FAILURE OR MALFUNCTION, CONTACT THE TECHNICAL SERVICE SRL FAZZINI OR QUALIFIED BY FAZZINI S.R.L. THE BATTERY MUST BE REPLACED BY QUALIFIED PERSONNEL ONLY

FAZZINI S.R.L. MAKES NO WARRANTY OF ANY KIND FOR EQUIPMENT THAT AFTER CHECKING THE TECHNICAL SERVICE BE TAMPERED. THE OPERATOR MUST REPLACE ONLY WHEN NECESSARY ACCESSORIES SUPPLIED IN LINE WITH THE INTENSITY 'USE OF EQUIPMENT THE USER CONTACTS THE FAZZINI S.R.L. IN CASE OF REPAIRS NEEDED.

SHIPPING COSTS MUST BE AGREED WITH THE FAZZINI SRL BEFORE SHIPMENT OF FAILURE.

IN CASE OF PENETRATION OF LIQUIDS INSIDE THE CONTACT OF A TECHNICAL SERVICE SRL FAZZINI FOR THE NEXT MAINTENANCE. NOT INTERVENE DIRECTLY ON THE DEVICE

STORAGE

THE ASPIRATOR MUST BE STORED IN A DRY LOCATION, PROTECTED FROM ATMOSPHERIC AGENTS

- STORAGE TEMPERATURE: BETWEEN – 10° AND +55° HR 20-80%
TRANSPORT TEMPERATURE – 10° AND +55° HR 20-80% AND VARIABLE HUMIDITY – 10° AND +55° HR 20-80%
DISPOSAL WITH REFERENCE TO THE DISPOSAL OF THE APPLIANCE, ITS ACCESSORIES AND ITS SINGLE USE DISPOSABLE PARTS, PLEASE SEE CURRENT WASTE DISPOSAL REGULATIONS DO NOT ACT DIRECTLY ON THE APPLIANCE

TO CHARGE DEVICE (F-18 BATTERY)

1. WHEN ON THE DEVICE'S DISPLAY ONLY THE RED LED IS ON, WHILE THE GREEN LED AND THE TWO YELLOW LED ARE OFF, IT IS NECESSARY TO DISCONTINUE OPERATIONS AND RECHARGE THE DEVICE.
2. TO CHARGE THE DEVICE, KEEP TO THE FOLLOWING INSTRUCTIONS: TO SWITCH POSITION "0", INSERT THE 230V PLUG INTO A WALL SOCKET, THE BLINKING LED SHALL TURN ON AND KEEP CHARGING FOR AT LEAST 12 HOURS.

THE DEVICE CAN OPERATE ON AN EXTERNAL 12 V BATTERY USING THE APPROPRIATE CABLE WITH CAR PLUG

HOW TO REPLACE THE BATTERY:

TURN OFF THE SUCTION PUMP BEFORE CUTTING THE CURRENT, UNSCREW THE 4 SCREWS ON ITS BELOW, BATTERY IS FIXED BY CLAMP INSIDE THE UNIT UNSCREW THE 2 SCREWS ON THE CLAMP

REMOVE THE BATTERY AND CONNECT THE NEW ONE ACCORDING TO PROPER POLARITY, POSITIVE POLARITY (+) BLUE OR RED WIRE / NEGATIVE POLARITY (-) BLACK WIRE, SCREW THE 2 SCREWS ON THE HOLDING CLAMP, CLOSE THE SUCTION UNIT SCREWING THE 4 SCREWS

DISPOSAL

WITH REFERENCE TO THE DISPOSAL OF THE APPLIANCE, ITS ACCESSORIES AND ITS SINGLE USE DISPOSABLE PARTS, PLEASE SEE CURRENT WASTE DISPOSAL REGULATIONS DO NOT ACT DIRECTLY ON THE APPLIANCE FAZZINI S.R.L. RESERVES ALL THE RIGHTS TO CHANGE THIS HANDBOOK WITH NO PRIOR NOTICE FURTHER TO MANUFACTURING CHANGES TO THE APPLIANCE, WHICH HAS BEEN BUILT IN COMPLIANCE WITH THE FOLLOWING STANDARDS: - CEI EN 60601/1 (CEI 62.5) - CEI EN 60601/1/2 - UNI EN ISO 10079/1 - DEVICE IN CONFORMITY WITH – 93/42 CEE & 2007/47/CE FAZZINI S.R.L. IS COMMITTED TO THE MONITORING OF THE MEDICAL APPLIANCES IT DISTRIBUTES ON THE MARKET, AND THEREFORE SHALL BE MOST GRATEFUL OF BEING INFORMED OF ANY PROBLEMS OR SUGGESTIONS RELATING TO ITS PRODUCTS, TO BE SENT IN WRITING TO: FAZZINI SRL S.S. PADANA SUP. 317 - 20090 VIMODRONE (MI) – ITALIA - FAX 0039/0227409242 - E-MAIL : FAZZINI@FAZZINI.IT - PLEASE STATE YOUR CONSENSUS TO DATA PROCESSING UNDER LAW 196/03

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